Important Information

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Additional information about wireless (RF) communication

Confirmation that a blood glucose test result has been received by your partner device

When the partner device is idle (at the Home Screen), it will beep or vibrate when it receives a blood glucose reading from the meter. The reading will appear on the partner device screen. See your partner device manual for more information.

Pairing (linking) your meter and Medtronic partner device

Programming your meter ID links your partner device to the meter. If you do not "link" the meter to your partner device, you will enter your blood glucose readings manually. Each meter has its own unique ID. This helps ensure only partner devices with your unique meter ID can automatically receive blood glucose test results from your meter. **Partner devices without your unique meter ID cannot receive your blood glucose test results.**

"Automatic" Transmission of Blood Glucose Test Results

- Blood glucose test results will be transmitted automatically if the PUMP COMM feature on your meter is turned on. Note that the PUMP COMM feature is turned on when you first receive your meter.
- If the PUMP COMM feature is turned off, your test results will not be transmitted automatically, and you will need to manually enter test results into your partner device.

Additional information about your meter and testing materials

Disposal of used test strips and lancets

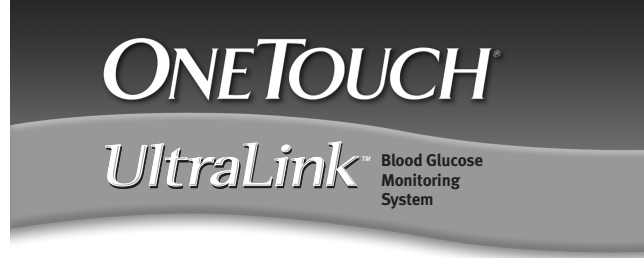
Used test strips and lancets may be considered biohazardous waste in your area. Be sure to follow your health care professional's recommendations or your local regulations for proper disposal.

Control solution

Use OneTouch® Ultra® Control Solution with your meter system. For information on a second level of control solution, contact Customer Service at 1 866 621-4846.

Additional Technical Specifications

Wireless (RF) operating range:	Maximum 4 feet*						
* Your Medtronic partner device must be within 4 feet of your meter to receive a blood glucose test result.							
Separation distance from other wireless (RF) devices:	Minimum 12 inches**						
** The use of cell phone within 12 inches of your meter or Medtronic partner device may interfere with the transmission of blood glucose test results to the partner device. Other types of wireless devices that are in use nearby may also prevent or delay the transmission of test results. Moving away from or turning off these interfering devices may allow communication.							
Expected delay (latency range) in wireless (RF) communication:	Minimum 3 seconds Maximum 60 seconds						
System accuracy: <i>Analytical Performance Characteristics</i> The accuracy of the OneTouch® UltraLink™ Blood Glucose Monitoring System was assessed by comparing blood glucose test results on 135 subjects, with those obtained using a YSI Model 2300 Glucose Analyzer. Six results were obtained for each subject (each tested in duplicate with test strips from 3 separate lots.) The following results were obtained: System Accuracy Results for Glucose Concentrations <75 mg/dL							
Within ±5 mg/dL	Within ±10 mg/dL	Within ±15 mg/dL†					
85/150 (56.7%)	137/150 (91.3%)	149/150 (99.3%)					
System Accuracy Results for Glucose Concentrations ≥75 mg/dL							
Within ±5%	Within ±10%	Within ±15%	Within ±20%†				
234/660 (35.5%)	464/660 (70.3%)	608/660 (92.1%)	653/660 (98.9%)				
System Accuracy Results across the entire Glucose Range							
Within ±15 mg/dL or ±20%							
802/810 (99.0%)							
These results indicate that the OneTouch® UltraLink™ Blood Glucose Monitoring System meets the ISO 15197 requirements for accuracy. † ISO 15197 Minimum Acceptable Accuracy Requirements: <ul style="list-style-type: none">• 95% of individual glucose results must fall within ±15 mg/dL of the YSI reference at glucose concentrations <75 mg/dL• 95% of individual glucose results must fall within ±20% of the YSI reference at glucose concentrations ≥75 mg/dL							
Regression Statistics							
# of Subjects	# of Tests	Slope (mg/dL)	Intercept (mg/dL)	95% CI Slope	95% CI Intercept	Std. Error (S _{y.x})	R ²
135	810	0.970	-2.77	(0.960, 0.979)	(-4.75, -0.80)	14.98	0.981
Samples were tested in duplicate on each of three test strip lots. Results indicate that the OneTouch® UltraLink™ Meter compares well with a laboratory method.							
Precision							
Within Run Precision (100 venous blood tests)				Total Precision (200 control solution tests)			
Target Glucose (mg/dL)	Mean Glucose (mg/dL)	Standard deviation (mg/dL)	Coefficient of variation (%)	Glucose level (mg/dL)	Mean Glucose (mg/dL)	Standard deviation (mg/dL)	Coefficient of variation (%)
40	50.78	1.56	3.07	Low	47.52	1.42	2.99
100	108.84	2.26	2.08	Normal	116.22	2.23	1.92
130	136.12	2.26	1.66	High	339.71	6.61	1.95
200	210.08	3.01	1.43				
300	305.66	4.07	1.33				
Results show that the greatest variability observed (of two lots tested) is 3.1% or less.							

Customer Service contact information

Customer Service is available 24 hours a day, 7 days a week, toll free at 1866 621-4846.

© 2008 LifeScan, Inc. Milpitas, CA 95035 USA
Rev. date: 04/2008
AW 06563301A

Important Information

about your OneTouch® UltraLink™ Blood Glucose Monitoring System

Please read this addendum carefully and keep it with the printed Owner's Booklet.

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