ONE TOUCH[®] TEST STRIPS

Ultra TIRAS REACTIVAS

TIRAS DE TESTE

For Testing Glucose in Whole Blood

IMPORTANT: Please read this information and your OneTouch[®] Ultra or InDuo[™] Brand Owner's Booklet before using OneTouch[®] Ultra Test Strips.

Intended Use

OneTouch[®] Ultra Test Strips are used with OneTouch[®] Ultra Brand and InDuo[™] Brand Blood Glucose Meters for quantitatively measuring glucose in whole blood. OneTouch[®] Ultra Brand and InDuo[™] Brand Meters read the glucose level of a whole blood sample on a OneTouch[®] Ultra Test Strip. OneTouch[®] Ultra Brand and InDuo[™] Brand Systems are plasma-calibrated to allow easy comparison of results with laboratory methods. OneTouch[®] Ultra Test Strips are for testing outside the body (*in vitro* diagnostic use).


Storage and Handling

- Store the test strip package in a cool, dry place not above 30°C (86°F). Keep away from direct sunlight and heat. Do not refrigerate.
- Store your test strips in their original vial only; do not transfer them to a new vial or any other container.
- After removing a OneTouch[®] Ultra Test Strip from the vial, immediately replace the vial cap and close it tightly.
- Use each test strip immediately after removing it from the vial.
- Write the discard date on the vial label when you first open it. Discard remaining OneTouch[®] Ultra Test Strips 3 months after first opening the vial.
- Avoid getting dirt or food on the top edge.
- Do not use test strips beyond the expiration date printed on the package since they may cause inaccurate results.
- Do not bend, cut, or alter a OneTouch[®] Ultra Test Strip in any way.

WARNING: Keep the test strip vial away from children; the cap is a choking hazard. Also, the vial or cap may contain drying agents that are harmful if inhaled or swallowed and may cause skin or eye irritation.

Precautions to Obtain Accurate Results

- Code the OneTouch[®] Ultra Brand or InDuo[™] Brand Blood Glucose Meter to match the code number printed on the OneTouch[®] Ultra Test Strip vial.
- Use only OneTouch[®] Ultra Test Strips with the OneTouch[®] Ultra Brand or the InDuo[™] Brand Meter to obtain accurate and consistent results.
- If you are experiencing symptoms that are not consistent with your blood glucose test results AND you have followed all instructions described in your OneTouch[®] Ultra Brand or InDuo[™] Brand Owner's Booklet, call your healthcare professional.
- Never make significant changes to your diabetes control program or ignore physical symptoms without speaking with your healthcare professional.

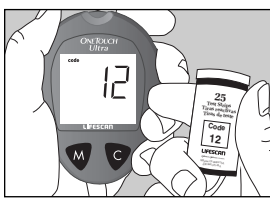(Example)

Test Procedure for Blood Glucose Measurement

Materials provided: OneTouch[®] Ultra Test strips.

Materials required but not provided: OneTouch[®] Ultra Brand or InDuo[™] Brand Blood Glucose Meter with Owner's Booklet; OneTouch[®] UltraSoft[™] Adjustable Blood Sampler with optional OneTouch[®] UltraClear[™] cap; new sterile lancet.

Blood Sample Collection

To obtain a drop of blood, follow these steps:

Step 1: Select the puncture site. You may obtain a blood drop from either a fingertip or an arm. If using an arm, select a soft, fleshy area (away from bone) that is free of visible veins or excess hair. Wash your hands and the puncture site with warm, soapy water. Dry them thoroughly.

Step 2: Lance the fingertip or arm. If lancing the fingertip: Hold the sampler firmly against the side of your finger. Press the release button. Massage the fingertip to obtain a round drop of blood. Do not squeeze the puncture site excessively. If lancing the arm: Replace the regular sampler cap with the OneTouch[®] UltraClear[™] cap and set the sampler for a deeper puncture. Massage the area gently. If necessary, rub the area more vigorously or apply heat briefly. Press and hold the sampler against the arm for a few seconds. Press the release button. Hold the sampler in place until a blood drop forms that will fill the confirmation window of the test strip. If your massage the area to obtain more blood, do not squeeze the site excessively.

The blood sample must be at least 1 µL in volume (● actual size) or you may get an inaccurate result.

IMPORTANT INFORMATION ABOUT ARM TESTING

- Under certain conditions, blood glucose test results obtained using samples taken from your arm may differ significantly from fingertip samples.
- The conditions in which these differences are more likely to occur are when your blood glucose is changing rapidly such as following a meal, an insulin dose or associated with physical exercise.
- When blood glucose is changing rapidly, fingertip samples show these changes more quickly than arm samples.
- When your blood glucose is falling, testing with a fingertip sample may identify a hypoglycaemic (low blood sugar) level sooner than a test with an arm sample.
- Use arm samples only for testing prior to, or more than two hours after, meals, insulin dose, or physical exercise.
- Testing performed within two hours after meals, an insulin dose, or physical exercise, or whenever you feel that your glucose levels may be changing rapidly, should be done from the fingertip.
- You should also use fingertip testing whenever you have a concern about hypoglycaemia (insulin reactions) such as when driving a car, particularly if you suffer from hypoglycaemic unawareness (lack of symptoms to indicate an insulin reaction), as arm testing may fail to detect hypoglycaemia.

What you should do:

- Use arm or fingertip samples for testing prior to, or more than two hours after, meals, insulin dose, or physical exercise.
- Routine testing before meals can be done either at the fingertip or the arm.
- Consult your healthcare professional before you begin using the arm for testing.

Blood Glucose Testing

Step 1: Insert test strip. Remove a test strip from the vial. Recap the vial immediately. Insert a test strip, contact bars end first and facing up, into the test port. Push it in until it will go no further. The meter will turn on and the display check will appear briefly. Then the code number will appear, followed by the Δ symbol with the unit of measure. Check that the unit of measure is set correctly. Be sure the meter and test strip codes match. If they do not, code the meter correctly.

Step 2: Apply sample. Using the OneTouch[®] UltraSoft[™] Adjustable Blood Sampler, obtain a blood sample large enough to fill the confirmation window. When the Δ symbol appears on the display, touch and hold the drop of blood to the narrow channel in the top edge of the test strip.

DO NOT:

- Apply sample to the front or back of the test strip.
- Push your finger against the test strip.
- Apply a smeared sample.

Hold the blood drop to the top edge of the test strip until the confirmation window is full before the meter begins to count down. If the confirmation window does not fill completely before the meter begins to count down, do not add more blood to the test strip; discard the strip and retest. If the confirmation window is not full you may get an Er 5 message or an inaccurate test result. If you have trouble filling the strip contact LifeScan Customer Services for assistance.

Step 3: Accurate results in just 5 seconds. Your blood glucose result will appear after the meter counts down from 5 to 1. Blood glucose test results are automatically stored in the meter memory. Turn the meter off by removing the test strip.

For detailed information on the test procedure and coding your meter, please refer to your OneTouch[®] Ultra Brand or InDuo[™] Brand Owner's Booklet.

Test Results

- Blood glucose test results are shown on the meter as either milligrams of glucose per deciliter of blood (mg/dL) or millimoles of glucose per litre of blood (mmol/L), depending on which unit of measure you have selected.
- OneTouch[®] Ultra Brand and InDuo[™] Brand Meters display results between 1.1 and 33.3 mmol/L (20–600 mg/dL). If your test result is lower than 1.1 mmol/L (20 mg/dL), "L" will appear on the meter display. This indicates severe hypoglycaemic (low blood glucose). You should immediately treat hypoglycaemia as recommended by your healthcare professional. If your test result is above 33.3 mmol/L (600mg/dL), "H" will appear on the meter display. This indicates severe hyperglycaemia (high blood glucose). You should seek immediate medical assistance.
- When your blood glucose result is above 13.3 mmol/L (240 mg/dL) or reads "H", "ketones?" will appear on the meter display. This message does not mean that the system detected ketones but that testing with a ketone strip may be advisable.

Range of Expected Values

Blood glucose monitoring requires the help of a healthcare professional. Together you can set your own range of expected blood glucose values, arrange your test times, and discuss the meaning of your blood glucose results. Expected blood glucose levels for people without diabetes:

Time	Range, mmol/L	Range, mg/dL
Before breakfast	3.9–5.8	70–105
Before lunch or dinner	3.9–6.1	70–110
1 hour after meals	Less than 8.9	Less than 160
2 hours after meals	Less than 6.7	Less than 120
Between 2 and 4 AM	Greater than 3.9	Greater than 70

If You Get Unexpected Results:

IMPORTANT: Low or high blood glucose readings can indicate a potentially serious medical condition. If your blood glucose reading is unusually low or high, or you do not feel the way your reading indicates, repeat the test with a new test strip. If your reading is not consistent with your symptoms or if your blood glucose result is less than 3.3 mmol/L (60 mg/dL) or higher than 13.3 mmol/L (240 mg/dL), you should contact your healthcare professional and follow his or her treatment advice.

Quality Control: Checking the System

Do a Control Solution Test:

- When you begin using a new vial of test strips
- At least once a week
- Whenever you suspect that the meter or test strips are not working properly
- When your blood glucose test results are not consistent with how you feel, or when you think your results are not accurate
- If you drop the meter

When control solution is applied to the top edge of the OneTouch[®] Ultra Test Strip, you should get results within the expected range printed on the test strip vial. If control solution test results fall outside this range, repeat the test. Results that fall outside the range may be caused by:

- Error in performing the test
- Failure to shake the control solution vial well enough (must shake vigorously)
- Expired or contaminated control solution
- Improper coding of the meter
- Test strip deterioration
- Meter malfunction
- Control solution that is too warm or too cool
- Failure to discard the first drop of control solution

IMPORTANT: If you continue to get OneTouch[®] Ultra Control Solution test results that fall outside the range printed on the vial, the OneTouch[®] Ultra Brand or your InDuo[™] Brand System may not be functioning properly. DO NOT use the system to test your blood until you get a control solution test result that falls within the range. If you continue to have problems, call LifeScan Customer Care Freephone: UK: 0800 121 200, Ireland: 1800 535 676.

LIMITATIONS

OneTouch[®] Ultra Test Strips give accurate results when the following limitations are observed:

- The test strips should not be used for the testing of newborns.
- The test strips are for single use only. Do not reuse.
- The test strips are specific to D-glucose and do not react to other sugars which may be present in blood.
- Use only fresh capillary whole blood. Do not use serum or plasma.
- Extremes in haematocrit may affect test results.² Haematocrit levels less than 30% may cause falsely high readings and haematocrit levels greater than 55% may cause falsely low readings. If you do not know your haematocrit level, consult your healthcare professional.
- OneTouch[®] Ultra Test Strips may be used at altitudes up to 10,000 feet without an effect on test results. Accurate results were demonstrated in clinical studies performed at altitudes up to 1,640 feet and in studies simulating altitudes up to 10,000 feet.
- In-Vitro Diagnosticum*
- Healthcare professionals—please note these additional limitations:**
- Fresh capillary blood may be collected into heparin-containing test tubes if the blood is used within 10 minutes. Do not use other anticoagulants or preservatives.
- Interferences: Acetaminophen, salicylates, uric acid, ascorbic acid (vitamin C), and other reducing substances (when occurring in normal blood or normal therapeutic concentrations) do not significantly affect results. However, abnormally high concentrations in blood may cause inaccurately high results.
- Patients undergoing oxygen therapy may yield falsely lower results.
- Test results may be falsely low if the patient is severely dehydrated, in shock, or in a hyperosmolar state (with or without ketosis). Critically ill patients should not be tested by blood glucose meters.
- Lipemic samples: Cholesterol levels up to 15.5 mmol/L (600 mg/dL) and triglycerides up to 4.55 mmol/L (400 mg/dL) do not affect the results. Grossly lipemic patient samples have not been tested and are not recommended for testing with OneTouch[®] Ultra Brand and InDuo[™] Brand Systems.

TEST PRINCIPLE

Glucose in the blood sample mixes with special chemicals on the test strip and a small electrical current is produced. This current is measured by the OneTouch[®] Ultra Brand or InDuo[™] Brand Meter and displayed as your blood glucose result. The strength of these currents changes with the amount of glucose in the blood sample.

REAGENT COMPOSITION

Each OneTouch[®] Ultra Test Strip contains: Glucose oxidase (*Aspergillus niger*) ≥ 0.8 IU; other ingredients (mediator, buffer, etc.) ≥ 0.05 mg. The vial or vial cap contains approximately 1.4 g silica gel or 2.8 g of molecular sieve.

PERFORMANCE CHARACTERISTICS

The performance of OneTouch[®] Ultra Test Strips has been evaluated both in laboratory and in clinical tests.²

Measurement range: The measurement range of the OneTouch[®] Ultra System and the InDuo[™] System is 1.1 to 33.3 mmol/L (20–600 mg/dL).

Accuracy: The accuracy of the OneTouch[®] Ultra System was assessed by comparing blood glucose results obtained by patients with those obtained using a YSI Model 2300 Glucose Analyzer, a laboratory instrument. The following results were obtained by 117 diabetic patients at 3 clinical centers:

Slope	0.986
y-intercept	-0.3 mmol/L
Correlation coefficient (r)	0.984
No. of samples	117
Range tested	2 — 24.1 mmol/L

This study shows that the OneTouch[®] Ultra System compares well with a laboratory method.

PRECISION:

Within Run Precision

Blood _{av}	2.5 mmol/L	CV = 3.2%
Blood _{av}	4.3 mmol/L	CV = 2.0%
Blood _{av}	7.2 mmol/L	CV = 2.1%
Blood _{av}	12.2 mmol/L	CV = 1.8%
Blood _{av}	20.2 mmol/L	CV = 1.6%

Total Precision

Control	2.4 mmol/L	CV = 4.4%
Control	9.5 mmol/L	CV = 2.6%
Control	20.3 mmol/L	CV = 2.4%

This study shows a variability from strip to strip in blood tests of 3.2% or less.

IMPORTANT: For a complete description of operating instructions and other important technical information, please refer to your OneTouch[®] Ultra Brand or InDuo[™] Brand Owner's Booklet. IF YOU HAVE QUESTIONS ABOUT THE USE OF ANY LIFE SCAN PRODUCT, PLEASE CONTACT LIFE SCAN CUSTOMER CARE FREEPHONE: UK: 0800 121 200, Ireland: 1800 535 676.

References

- Krall, L.P., and Beaser, R.S.: Joslin Diabetes Manual. Philadelphia: Lea and Febiger (1989), 138.
- Data on file.

OUR COMMITMENT TO YOU: Our goal is to provide you with quality healthcare products and dedicated customer service. If you are not fully satisfied with this product, we offer a No-Risk, Money-Back Guarantee and will refund your purchase within 30 days. Please contact us regarding a refund or with questions about the use of any LifeScan product. For assistance, please contact LifeScan Customer Care Freephone: UK 0800 121 200, Ireland: 1800 535 676.

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LIFESCAN
a Johnson & Johnson company
Milpitas, California 95035, USA

LIFESCAN		APPLIED GRAPHICS		PRODUCTION NOTES	
AGT Job No.	-0012573-08	Marketing Associate	-Lucinda Leung	Die No.	-
Artwork No.	-060-379-01A	Phone Extension	-4161	Folds	-YES
Round No. & Date	-eight 12-11-01	Product Variant	-Ultra	Varnish	-NO
DCO & Date	-	Marketing Country(ies)	-EPM	No. of Colors	-1
Project	-OneTouch	CPS Reference	-050-654-01	Artwork program	-QuarkxPress 4/Illustrator 8
Component	-TSI- GB,ES,PT	Printer	-Bantia	Type of Locked file	-
SPOT COLORS					
 Black					
PANTONE® is a registered trademark of Pantone, Inc. All information contained herein is the CONFIDENTIAL property of Johnson & Johnson and may not be duplicated or released without the expressed written permission of LifeScan.					
AGT_rev1.8_10101					

PRODUCTION NOTES

AGT Desktop SW
Amended by/Date -

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© LifeScan, Inc. 2000