

## **Acceptable Cleaning Agents for use with SureStep® Flexx® Meters**

Dear LifeScan Customer:

The following compounds have been tested and may be used to clean the exterior of SureStep® Flexx® Meters. Product names may be trademarks of their respective owners.

<b>Water</b>	<b>70% isopropyl alcohol</b>
<b>Cavicide®</b>	<b>Progard™</b>
<b>Hibiclens®</b>	<b>Staphene®</b>
<b>10% bleach</b>	<b>12.5% ammonia</b>
<b>Cidex® (glutaraldehyde)</b>	<b>Softgard™</b>
<b>22% Lysol®</b>	<b>5.5% WEX-Cide™</b>
<b>Gluco-Chlor wipes</b>	<b>10% Bacdown®</b>
<b>Cidex® Plus</b>	<b>CaviWipes™ (17.2% Isopropanol)</b>
<b>Sanicloth™ (55% isopropanol)</b>	

- Do not use alcohol, cleaners containing alcohol, glass cleaners, or any cleansers containing abrasives, phenol, or ammonia to clean the test strip holder or lens area because it will damage the SureStep® Flexx® Meter parts.
- Do not use cleaning agents that contain hydrogen peroxide or accelerated hydrogen peroxide (e.g. Accel with Accelerated Hydrogen Peroxide™ by Virox Technologies, Inc.; Oxivir™ Tb Cleaner Disinfectant by JohnsonDiversey).
- Do not leave SureStep® Flexx® Meters in the room during decontamination if products containing hydrogen peroxide or accelerated hydrogen peroxide are used for decontamination (e.g. Bioquell).

**The use of cleaning agents containing hydrogen peroxide or accelerated hydrogen peroxide on or near SureStep® Flexx® Meter may lead to an error message or an inaccurate, high result.**

Refer to the SureStep® Flexx® Operator's Guide for cleaning precautions and detailed information on cleaning the meter and test strip holder.

Please call **LifeScan's Healthcare Professional Line** at **1 800-524-7226** if you have additional questions regarding proper cleaning procedures and acceptable cleaning agents for SureStep® Flexx® Meters.