

# Updates to IN TOUCH® Diabetes Management Software (Version 1.33)

**PLEASE KEEP THIS INFORMATION WITH YOUR USER'S GUIDE**


## Additions/Corrections to the IN TOUCH® Software User's Guide

- IN TOUCH Diabetes Management Software now supports the ONE TOUCH® Ultra Meter and the InDuo™ Meter. This will allow you to download your blood glucose readings from your Ultra Meter or InDuo Meter to IN TOUCH Software so that you can track and review your blood glucose levels.
- IN TOUCH Software now supports Windows 2000.
- IN TOUCH Software no longer supports Windows 3.1.


## Connecting the ONE TOUCH Ultra Meter or the InDuo Meter to Your Computer

Use the LifeScan Interface Cable to connect your ONE TOUCH Ultra Meter or your InDuo Meter to your computer.


- 1 Turn off the meter.
- 2 Make sure the COM port selection on the IN TOUCH Preferences screen matches the computer COM port to which you are connecting the Interface Cable.
- 3 Connect the Interface Cable to the serial port on your computer.

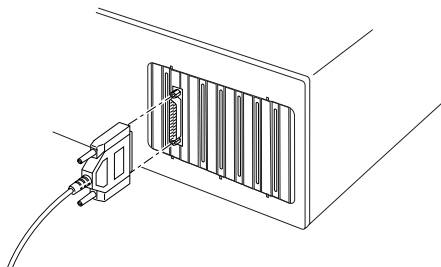

- 4 Connect the stereo jack on the Interface Cable to the meter's data port.

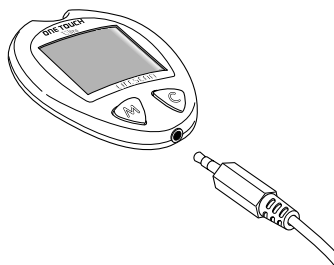

ONE TOUCH Ultra Meter

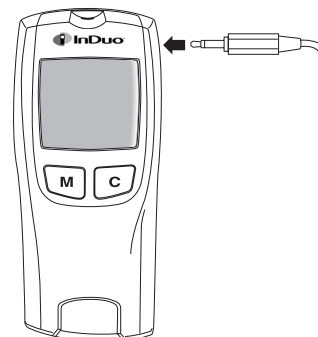

InDuo Meter

For detailed information on using IN TOUCH Software, refer to the *IN TOUCH Diabetes Management Software User's Guide*.