

# ONETOUCH®

## Diabetes Management Software v2.3




## Quick Reference Guide


Look inside for step-by-step instructions on how to:

- » Transfer readings from your OneTouch® Meter
- » View and share any of the 11 available reports
- » Add information to a patient record
- » Add or edit information related to a glucose reading


# OneTouch® Diabetes Management Software (DMS) v2.3 works with the entire family of OneTouch® Meters.

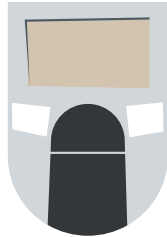


OneTouch® Basic®  
OneTouch® Basic® Plus  
OneTouch® II  
OneTouch® Profile®

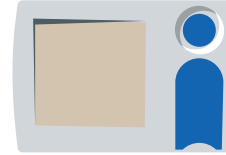

OneTouch® SureStep®  
OneTouch® SureStep® Plus  
OneTouch® GlucoTouch®  
OneTouch® GlucoTouch® Plus

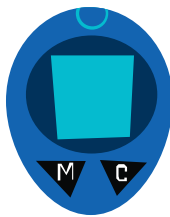

OneTouch® Ultra®  
OneTouch® Ultra®2  
OneTouch® FastTake®  
OneTouch® SmartScan™  
OneTouch® EuroFlash®  
OneTouch® PocketScan®

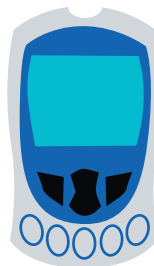


OneTouch® UltraSmart™



InDuo®

If you don't see your meter listed, refer to the Owner's Booklet for that meter to check if it is supported by this version of OneTouch® DMS.



## OneTouch® DMS v2.3 has 11 reports to manage the information that can help you improve your glycemic control.

- » Track glucose levels along with health-related data that can impact readings. This report lets you easily spot trends relative to a particular meal slot.

**Logbook**

- » Identify patterns in glucose readings by time of day.

**Standard Day**

- » Rank patients by key measures to flag those who need closer management.

**Exception**

- » View a summary of key reports (Logbook Report, Pie Chart Report, and Glucose Trend Report) for a given 14-day date range.


**Summary**

- » View all data in the software database (glucose, medications, food, exercise) in sequential format.

**Data List**

- » Analyze the relationship between glucose readings, insulin dosing, and carb intake.

**Insulin**



## OneTouch® DMS v2.3 has 11 reports to manage the information that can help you improve your glycemic control (continued).

- » Display the percentage of glucose readings that fall above, below and within target range for each meal slot.

### Pie Chart

- » Easily monitor how other health factors (blood pressure, weight, doctor visits, and HbA1c) are impacted by diabetes.

### Health Checks

- » Track changes in glucose readings from day to day.

### Glucose Trend

- » Identify before and after meal patterns across a distribution of glucose ranges.

### Histogram

- » Monitor how glucose readings are impacted by meals, exercise, and daily activities.

### Average Reading



## Get your OneTouch® Meter ready to transfer readings.

» To begin data transfer, the meter you are connecting may need to be turned **ON** or **OFF** depending on the meter type. Find your meter on one of the lists below and follow the **ON** or **OFF** instructions.

» These meters should be turned **OFF** before downloading:

- OneTouch® *FastTake*® Meter
- OneTouch® *PocketScan*® Meter
- OneTouch® *EuroFlash*® Meter
- OneTouch® *SmartScan*™ Meter
- OneTouch® *Ultra*® Meter
- OneTouch® *Ultra*®2 Meter
- *InDuo*® Meter

» These meters should be turned **ON** before downloading:

- OneTouch® *Basic*® Meter
- OneTouch® *Basic*® *Plus* Meter
- OneTouch® *Profile*® Meter
- One Touch® *II* Meter
- OneTouch® *SureStep*® Meter
- OneTouch® *SureStep*® *Plus* Meter
- OneTouch® *GlucoTouch*® Meter
- OneTouch® *GlucoTouch*® *Plus* Meter
- OneTouch® *UltraSmart*™ Meter

If you don't see your meter listed, refer to the Owner's Booklet for that meter for instructions on whether to turn the device **ON** or **OFF** before communicating.

**Note:** Meters that need to be turned **ON** for data transfers may turn off if there is a delay in connecting the meter. If this happens, just turn the meter back **ON**.



# Transfer OneTouch® Meter readings for viewing.

OneTouch® DMS makes it easy to transfer OneTouch® Meter readings for viewing.


## Step 1

Double-click on the desktop icon to open the Software.



## Step 2


Click on the **Download Meter Readings** icon on the Home Screen.

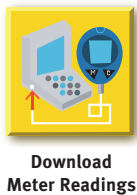


## Step 3

Connect the meter to the PC.

- » Plug the OneTouch™ Interface Cable into the COM or USB port on your PC.
- » Plug the other end of the OneTouch™ Interface Cable into your meter.
- » If you are using the OneTouch® *FastTake*®/*PocketScan*®/*EuroFlash*®/*SmartScan*® Meter, you must plug the OneTouch™ Interface Cable into the meter adapter. Then insert the adapter strip into the meter test strip port.

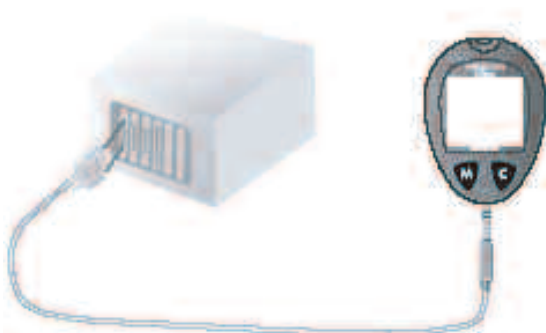



# Transfer OneTouch® Meter readings for viewing (continued).

## Step 4

To begin data transfer, make sure the meter you are connecting is turned **ON** or **OFF**, depending on the meter type. See *Get your OneTouch® Meter ready to transfer readings*.

## Step 5

Click on the **Download Meter Readings** button.




Download Meter Readings

## Step 6

Your Favorite Report will automatically open.

- » Customize your Favorite Report by selecting Preferences/Report Options from the Menu bar.

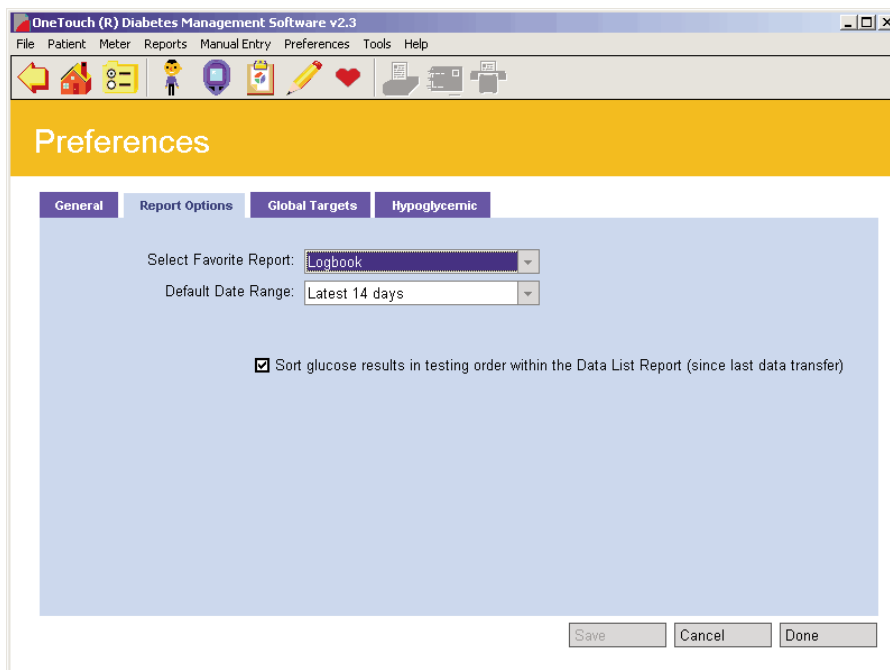



# View and share reports.

View and share reports with just a few clicks.

## Step 1

Double-click on the desktop icon to open the Software.



## Step 2


Click on the **View Reports** icon on the Home Screen.



View Reports

## Step 3

Click on the report you would like to view.

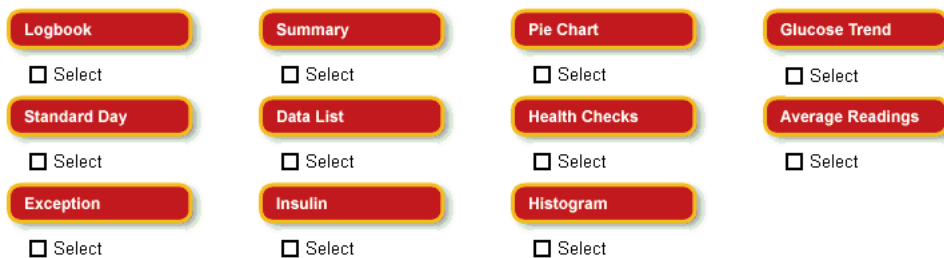

## Step 4

Share the report with another person.

Click on the **print**, **e-mail**, or **fax** icon in the tool bar, and follow the directions.





# Add information to a patient record.

Customize your personal diabetes profile.

- » General information
- » Schedules
- » Medications
- » Insulin regimens
- » Complications

## Step 1

Double-click on the desktop icon to open the Software.



## Step 2


Click on the **Set-Up** icon on the Home Screen.



## Step 3

Click on the **Patient Setup** icon on the Setup Screen.





# Add information to a patient record (continued).


## Step 4

Choose the correct name from the drop-down menu.

**Patient**  ▼

## Step 5

Use the blue tabs to add relevant information.

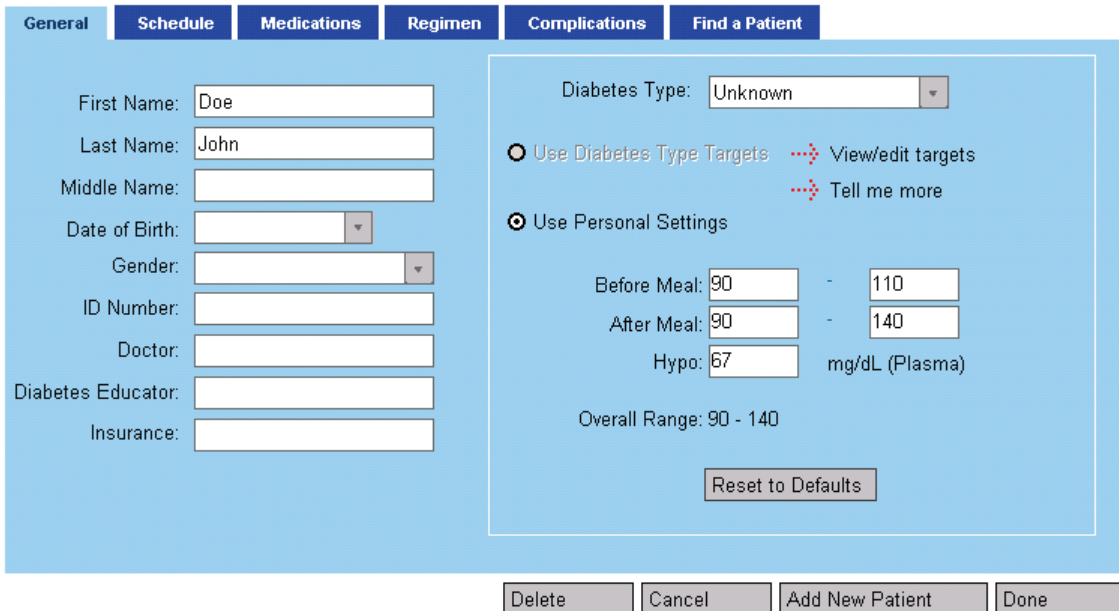


The screenshot shows a patient information form with several tabs: General, Schedule, Medications, Regimen, Complications, and Find a Patient. The 'Complications' tab is selected and highlighted in blue. The form contains the following fields and options:

- General Information:**
  - First Name:
  - Last Name:
  - Middle Name:
  - Date of Birth:  ▼
  - Gender:  ▼
  - ID Number:
  - Doctor:
  - Diabetes Educator:
  - Insurance:
- Diabetes Settings (under the 'Complications' tab):**
  - Diabetes Type:  ▼
  - Use Diabetes Type Targets [View/edit targets](#) [Tell me more](#)
  - Use Personal Settings
  - Before Meal:  -
  - After Meal:  -
  - Hypo:  mg/dL (Plasma)
  - Overall Range: 90 - 140
  -

At the bottom of the form, there are four buttons: Delete, Cancel, Add New Patient, and Done.

## Step 6

Click on the **Done** button to save changes.



Add or edit information related to glucose readings.


## Step 1

Double-click on the desktop icon to open the Software.



## Step 2

Click on the **View Reports** icon on the Home Screen.

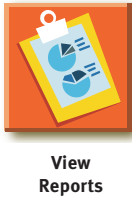

## Step 3


Click on the **Logbook** button on Choose Your Report.



## Step 4

Click on the report reading you would like to change.


122
175



## Add or edit information related to glucose readings (continued).

### Step 5

Add or edit data as needed in the blue charts.  
Use the tabs to move from section to section.

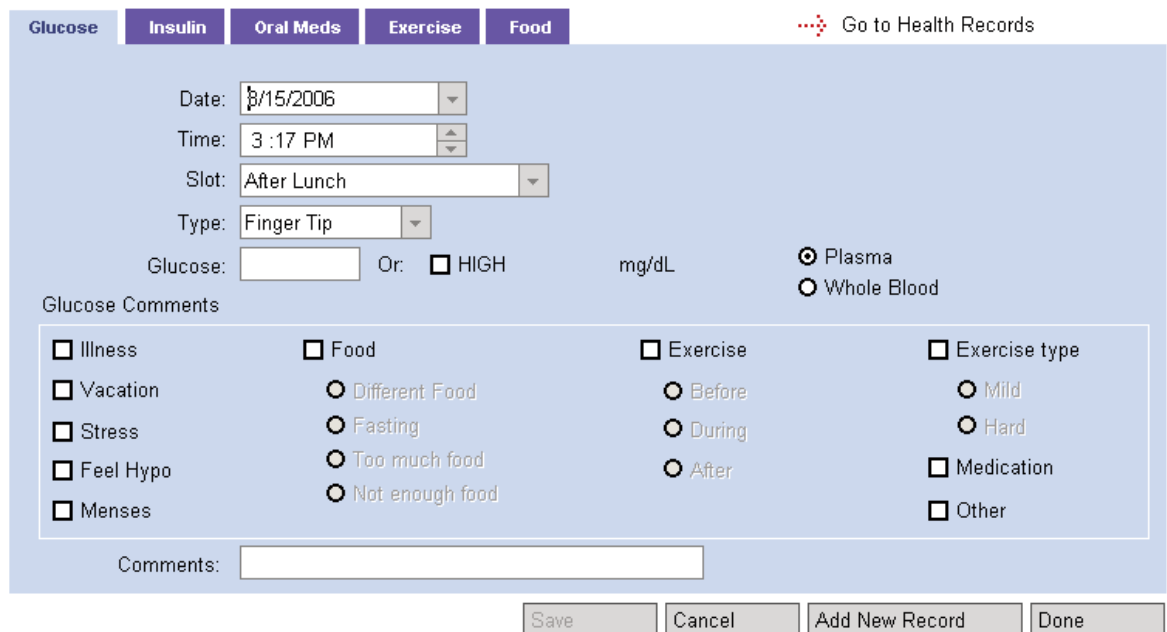

Glucose Insulin Oral Meds Exercise Food [Go to Health Records](#)

Date: 3/15/2006  
Time: 3:17 PM  
Slot: After Lunch  
Type: Finger Tip

Glucose:  Or:  HIGH mg/dL  Plasma  Whole Blood

Glucose Comments


<input type="checkbox"/> Illness	<input type="checkbox"/> Food	<input type="checkbox"/> Exercise	<input type="checkbox"/> Exercise type
<input type="checkbox"/> Vacation	<input type="radio"/> Different Food	<input type="radio"/> Before	<input type="radio"/> Mild
<input type="checkbox"/> Stress	<input type="radio"/> Fasting	<input type="radio"/> During	<input type="radio"/> Hard
<input type="checkbox"/> Feel Hypo	<input type="radio"/> Too much food	<input type="radio"/> After	<input type="checkbox"/> Medication
<input type="checkbox"/> Menses	<input type="radio"/> Not enough food		<input type="checkbox"/> Other

Comments:

Save Cancel Add New Record Done

### Step 6

Click the **Done** button to save the changes.

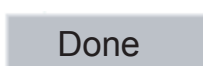



## General Troubleshooting.

If you have difficulty using OneTouch® DMS please consult the appropriate section of the user manual and help file. If that does not resolve the problem, please call or email LifeScan for OneTouch® DMS technical support.

» België:	service@lifescanbenelux.com	0800-15325
» Belgique:	service@lifescanbenelux.com	0800-14660
» Canada:	CustomerService@LifeScanCanada.com Servicealaclientele@LifeScanCanada.com	1 800 663-5521
» Deutschland:	service@lifescan.de	0800-72 55 007
» España:	info@lifescan.es	900 100 228
» France:	info@LifeScanFrance.com	0800 459 459
» Grand-Duché de Luxembourg:	service@lifescanbenelux.com	0800-25749
» Ireland:	technicalsupport@lifescan.co.uk	1800 535676
» Nederland:	service@lifescanbenelux.com	0800-0222445
» Österreich:	kundenservice@lifescan.at	0800 244 245
» Schweiz/ Suisse/ Svizzera:	info@LifeScan.ch	0800-850 905
» United Kingdom:	technicalsupport@lifescan.co.uk	0800 028 0616
» United States:	CustomerService@LifeScan.com	1 800 382-7226

**For all other countries:** contact your local LifeScan Office or visit the Around the World section of the LifeScan website at [www.lifescan.com](http://www.lifescan.com) for the Technical Support telephone number in your area.

## Use in Partnership with Your Healthcare Professional.

OneTouch® DMS is designed to help individuals and healthcare professionals track, review, and monitor key data related to managing diabetes. OneTouch® DMS is not intended to provide medical advice. If you have diabetes, consult with your healthcare professional before making any changes to your lifestyle, medication, diet, or exercise program.



LifeScan, Inc.,  
Milpitas, CA  
95035 USA

LifeScan Regulatory Affairs Europe  
Division of Ortho-Clinical Diagnostics France  
1, rue Camille Desmoullins – TSA 40007  
92787 Issy-les-Moulineaux, Cedex 09  
France