

Logbook

Patient

Report mg/dL (Plasma)

Date Range

From To

Click on a date, value, or icon to view/edit information.

Date ▼	Breakfast					Lunch					Dinner					Night			Cmnt
	Bef	Aft	Meds	Carb	Other	Bef	Aft	Meds	Carb	Other	Bef	Aft	Meds	Carb	Other	Gluc	Meds	Carb	
12/08/02	68	131	20M	30							151		10M	60	X				
12/08/02											129								
12/07/02	172		20M	30		140			15		179		10M	45					
12/06/02	114		20M	30		88			15		144		10M	60		165		15	
12/05/02	264		20M	15	♥	199	143		30	X			10M	120	X				
12/04/02		165	20M	30	♥						309		4A	30					
12/04/02											223		5A						
Average	137	154				164	146				201	232				133			
In Target	40%	86%				43%	88%				27%	33%				71%			
# Results	15	7				14	8				11	3				7			

View Statistics & targets

Hypoglycemic Above Target Below Target

M=Premixed A=Rapid

Summary





Patient Report mg/dL (Plasma)
 Date Range From To

Click on a date for more information.

Date	Breakfast					Lunch					Dinner					Night		
	Bef	Alt	Meds	Carb	Other	Bef	Alt	Meds	Carb	Other	Bef	Alt	Meds	Carb	Other	Gluc	Meds	Carb
12/08/02	68	131	20M	30							151	10M	60		X			
12/08/02											129							
12/07/02	172		20M	30		140			15		179	10M	45					
12/06/02	114		20M	30		88			15		144	10M	60		165		15	
12/05/02	264		20M	15	♥	399	143		30	X		10M	120		X			
12/04/02		165	20M	30	♥						309	4A	30					
12/04/02											223	5A						
Average	137	154				164	146				201	232				133		
In Target	40%	86%				43%	88%				27%	33%				71%		
# Results	15	7				14	8				11	3				7		

View Statistics & targets

■ Hypoglycemic
■ Above Target
■ Below Target
M=Premixed A=Rapid

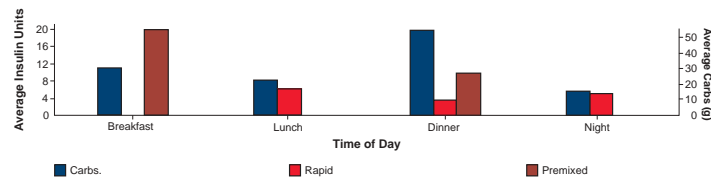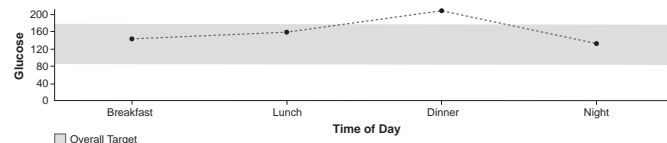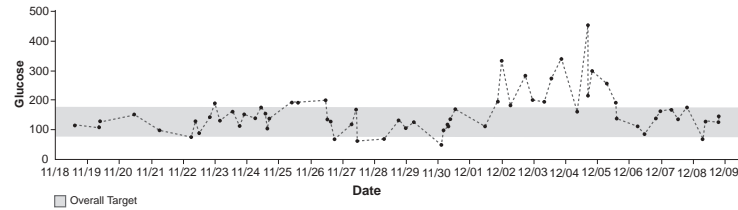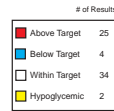

Pie Chart

Patient

Report


mg/dL (Plasma)

Date Range

From

To


Overall Totals

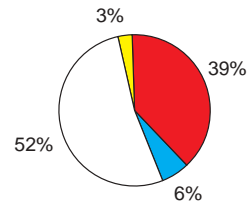

of Results

■ Above Target	25
■ Below Target	4
■ Within Target	34
■ Hypoglycemic	2


Before-breakfast

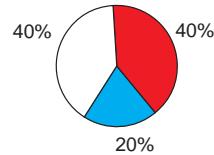

of Results

■ Above Target	6
■ Below Target	3
■ Within Target	6
■ Hypoglycemic	0


After-breakfast

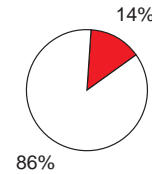

of Results

■ Above Target	1
■ Below Target	0
■ Within Target	6
■ Hypoglycemic	0


Before-lunch

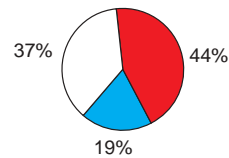

of Results

■ Above Target	7
■ Below Target	3
■ Within Target	6
■ Hypoglycemic	0

After-lunch

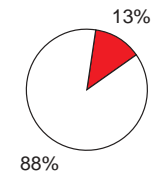

of Results

■ Above Target	1
■ Below Target	0
■ Within Target	7
■ Hypoglycemic	0

Glucose Trend


Patient ▼

Report ▼ mg/dL (Plasma)

Date Range ▼

From ▼

To ▼

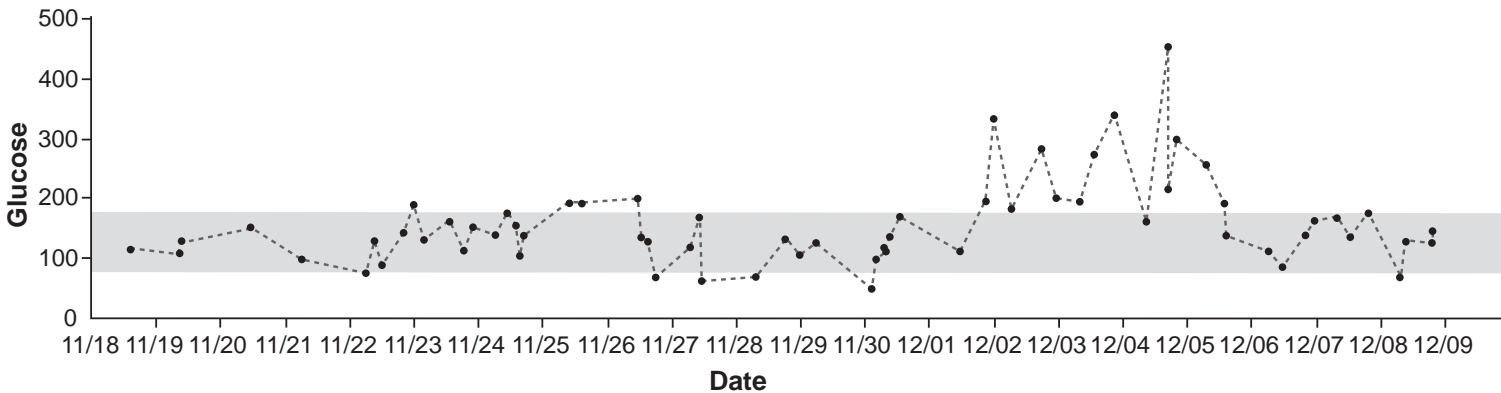

Overall Target


Standard Day

Patient ▼

Report ▼ mg/dL (Plasma)

Date Range ▼

From ▼ To ▼

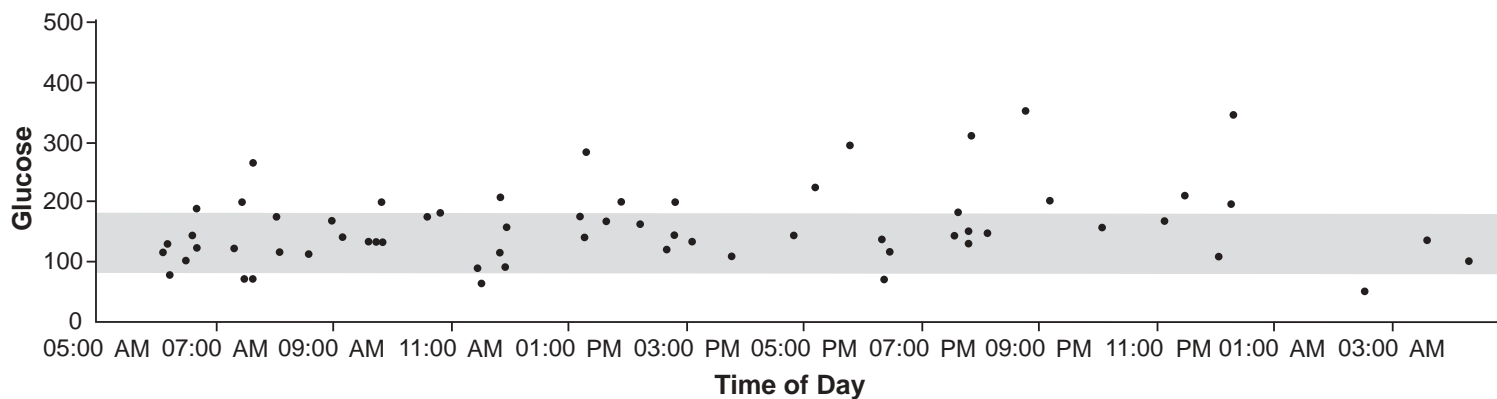

Overall Target

 View Statistics & Targets

Data List

Patient

Report mg/dL (Plasma)

Date Range

From To

Show Meter Serial Number

Click on a Date or Time or Comment Column to view/edit information.

Date ▼	Time	Slot	Result Type	Value ▼	Comments	Status
12/8/2002	07:50 PM	Before Dinner	Fats	10		
12/8/2002	07:50 PM	Before Dinner	Glucose	129		
12/8/2002	07:50 PM	Before Dinner	Carbs	60		
12/8/2002	07:50 PM	Before Dinner	Glucose	151		
12/8/2002	09:38 AM	After Breakfast	Glucose	131		
12/8/2002	08:01 AM	Before Dinner		01:00	Moderate, cleaning	Added

Hypoglycemic
 Above Target
 Below Target
 View Statistics & targets

Health Checks

Patient ▼

Report ▼ mg/dL (Plasma)

Date Range ▼

From ▼ To ▼

Result Type	Value	Date
Office Visit		11/19/2002
Height	65	11/19/2002
Weight	112	11/19/2002
Blood Pressure	129/71	11/19/2002
HbA1c	6.8%	11/19/2002
Cholesterol – Total	167	11/19/2002
Cholesterol – LDL	111	11/19/2002
Cholesterol – HDL	49	11/19/2002
Triglycerides	135	11/19/2002
Microalbumin	Normal	11/19/2002
Ketones	Negative	11/19/2002
Foot Exam		11/19/2002
Eye Exam		8/22/2002

Note: This listing includes the most recent results for each result type. Click on the result type to see past results

Average Reading



Patient

Report mg/dL (Plasma)

Date Range

From

To

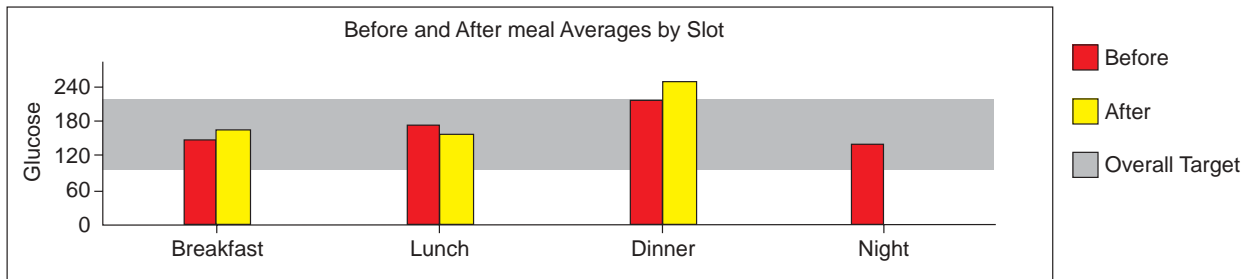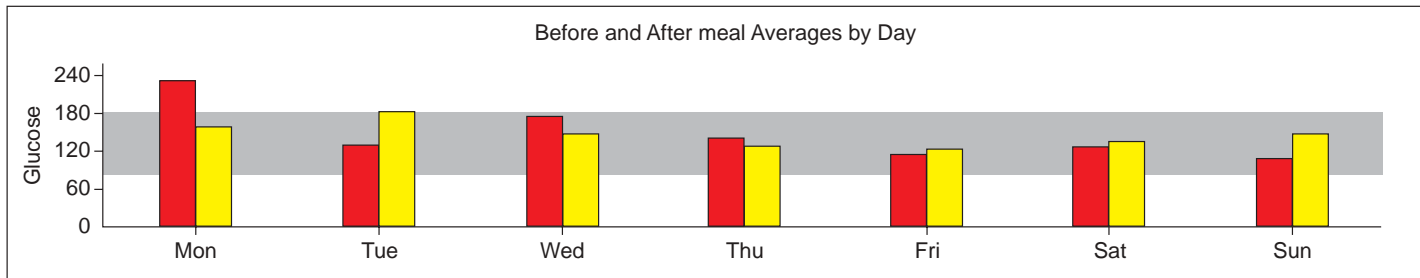

No date available for exercise glucose averages.

[View Statistics & Targets](#)

Insulin


Patient

Report mg/dL (Plasma)


Date Range

From

To

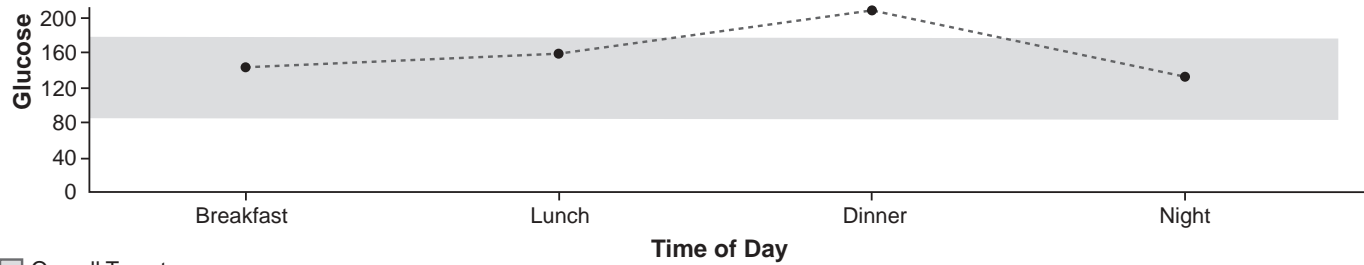

Overall Target

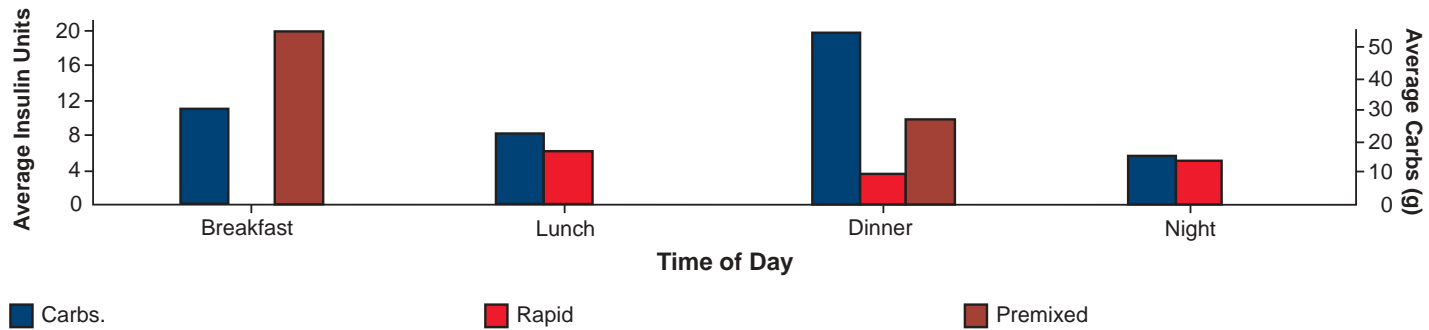

Histogram


Patient

Report mg/dL (Plasma)

Date Range

From

To

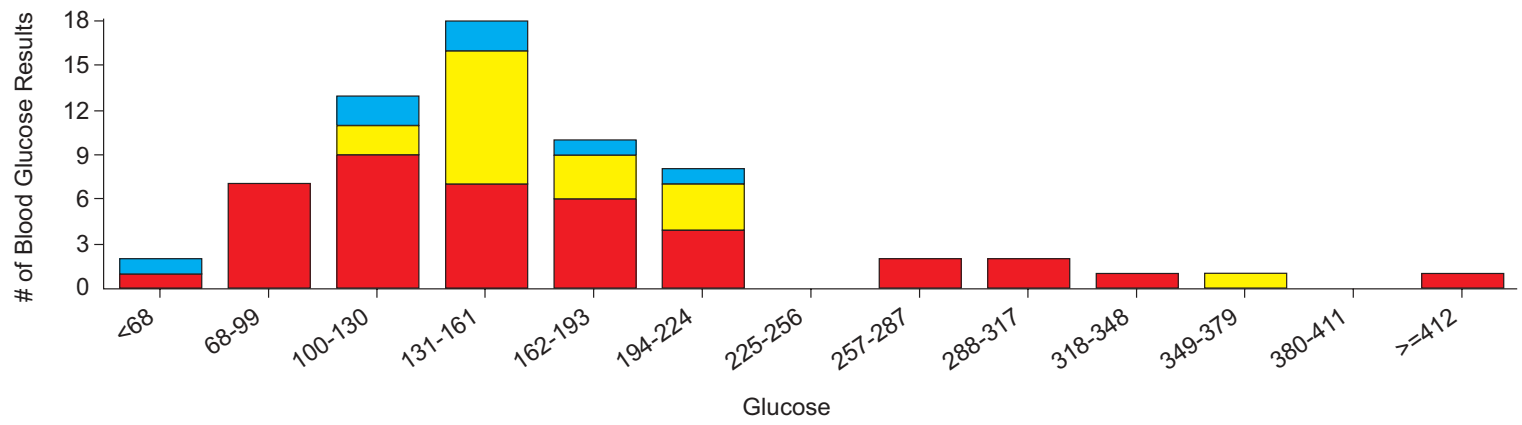

[View Statistics & Targets](#)

Before Meal After Meal Night

Exception

Group by insurance BC/BS

Report Exception mg/dL (Plasma)

Date Range Since last transfer

From 11/18/2002 To 12/08/2002 Apply

Patient ▲	# of Readings ▼	% Below Target ▼	% Above Target ▼	# Hypo ▼	HbA1c		Last Visit		
					Result ▼	Date ▼	Dr. ▼	Eye Exam ▼	Foot Exam ▼
1, Art	52	35	42	10					
3, Jenn	136	46	23	33	5.5%	9/11/2002	9/11/2002	8/22/2002	9/11/2002
4, Barbara	61	62	16	20					